
MORZELLATOR

MORCELLATOR

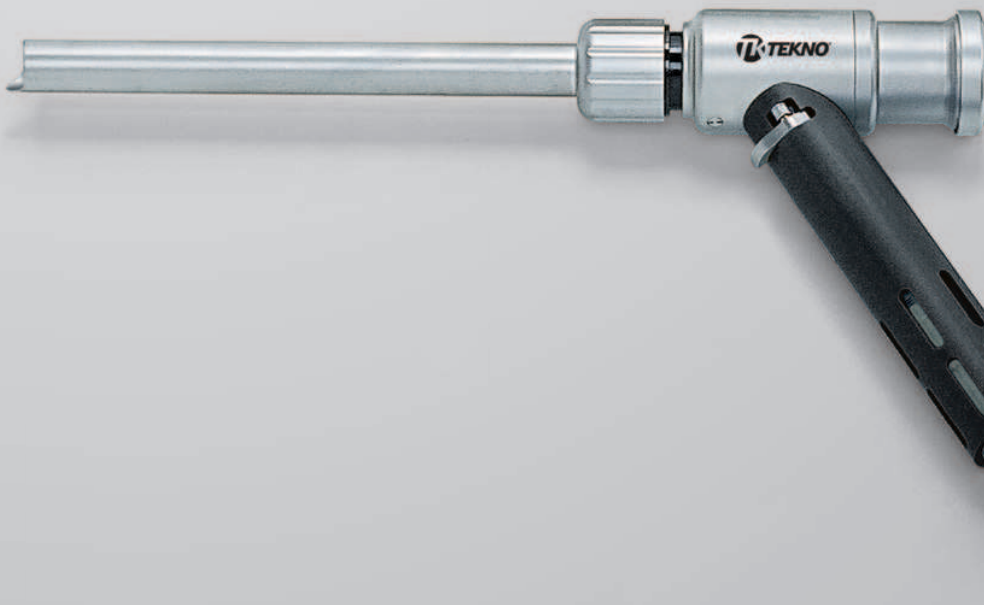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Nowadays newly developed medical devices make hysterectomy possible by a laparoscopic intervention. The advantage for the patient is less pain and a much faster recovery. Patients appreciate much the minimal invasive character of this operation because it leaves almost no scars on the patient's skin, reduces the healing of the wounds and saves money and time for all involved parties.

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LAPAROSCOPIC ACCESS

A manipulator (guide instrument) is inserted into the uterus via the vagina to enable it to be moved during the operation. A ceramic cap over the cervix later allows it to be safely withdrawn from the vagina. The abdominal cavity is insufflated with gas (CO₂) through a thin cannula in order to elevate the abdominal wall and provide an unobstructed view of the abdominal organs. The endoscop is inserted in the umbilical area through a percutaneous access. The required surgical instruments (e.g. electric scissors, morcellator, haemostatic clamps, etc.) are inserted in the lower abdomen through small (5 - 10 mm) incisions. The camera passes the image to a large monitor. The modern optics and instruments provide an unobstructed view and so ensure safe surgery

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INCISION

If the uterus requires complete removal (up to the outer cervix), the incision is made at the ceramic cap and then it is sutured. The severed uterus is cut from the abdominal cavity piece by piece with the morcellator and aspirated. In the case of a supracervical hysterectomy the uterus is cut above the inner cervix. Depending upon the diagnostic findings the physician opts either to remove the uterus through the vagina or by laparoscopic means. In the event of a higher risk, for example, due to an enlarged uterus and thus reduced visibility and constricted space, the surgery can be concluded by means of an abdominal incision. The uterus can, however, invariably be removed using morcellation in all the variants on a hysterectomy.