TPL

Tympanoplastik <i>Tympanoplasty</i>	4
Stapes Zusatzsieb <i>Additional stapes set</i>	8
Komplette Siebe <i>Complete sets</i>	9
Bohrsystem und Zubehör <i>Drilling system and accessories</i>	10

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Dear Colleagues,

Have you also experienced the disappointment of searching for a particular instrument, only to end up wondering when you finally had it in your hands whether it was the one you really wanted?

As a colleague in the field of ear, nose and throat surgery you will have been accustomed to a high standard of instruments in your clinical training and will be eager to maintain this standard in your own practice. Unfortunately, however, some of these standard instruments appear to have changed during the course of time. Even if an instrument is manufactured by the same company, it is frequently „not what it used to be“. Managing to find exactly the right instrument among the wide variety on offer is a tedious and time-consuming task which is aggravated by the fact that e.g. the „Cottle speculum“ you acquired is not the same as the one you had grown to appreciate in your many years of clinical practice.

Extra time and resources are again needed to have the instrument adapted to the desired and required expectations. In some cases subsequent modification is not even possible.

All the more welcome, then, is the news that Tekno Medical is doing something to counteract this trend. In cooperation with ENT surgeons, Tekno Medical is marketing instruments which have proved their reliability in many years of practice. Certain instruments are accompanied by a brief commentary outlining their outstanding features. In view of the minute dimensions involved in high-precision ENT work in body cavities, particular attention has been paid to the ultrafine design and slimline finish of the instruments.

In this catalogue series you will find individual sets of preselected instruments for the nose, paranasal sinuses and ear, which enable trouble-free performance of functional-aesthetic nose surgery, endonasal-endoscopic paranasal sinus surgery and middle-ear surgery including stapes surgery. The instruments have been selected on the basis of extensive operating experience in one of Germany's largest ENT clinics (Erlangen) and in my hospital practising.

If you have any questions or suggestions concerning the instruments, Tekno Medical and I are at your service at all times. Only a united effort will enable us to maintain and enhance our high standard of ENT surgery. Good instruments are an indispensable factor in achieving this aim. On this positive note, we wish you much enjoyment and pleasure working with these new instruments.

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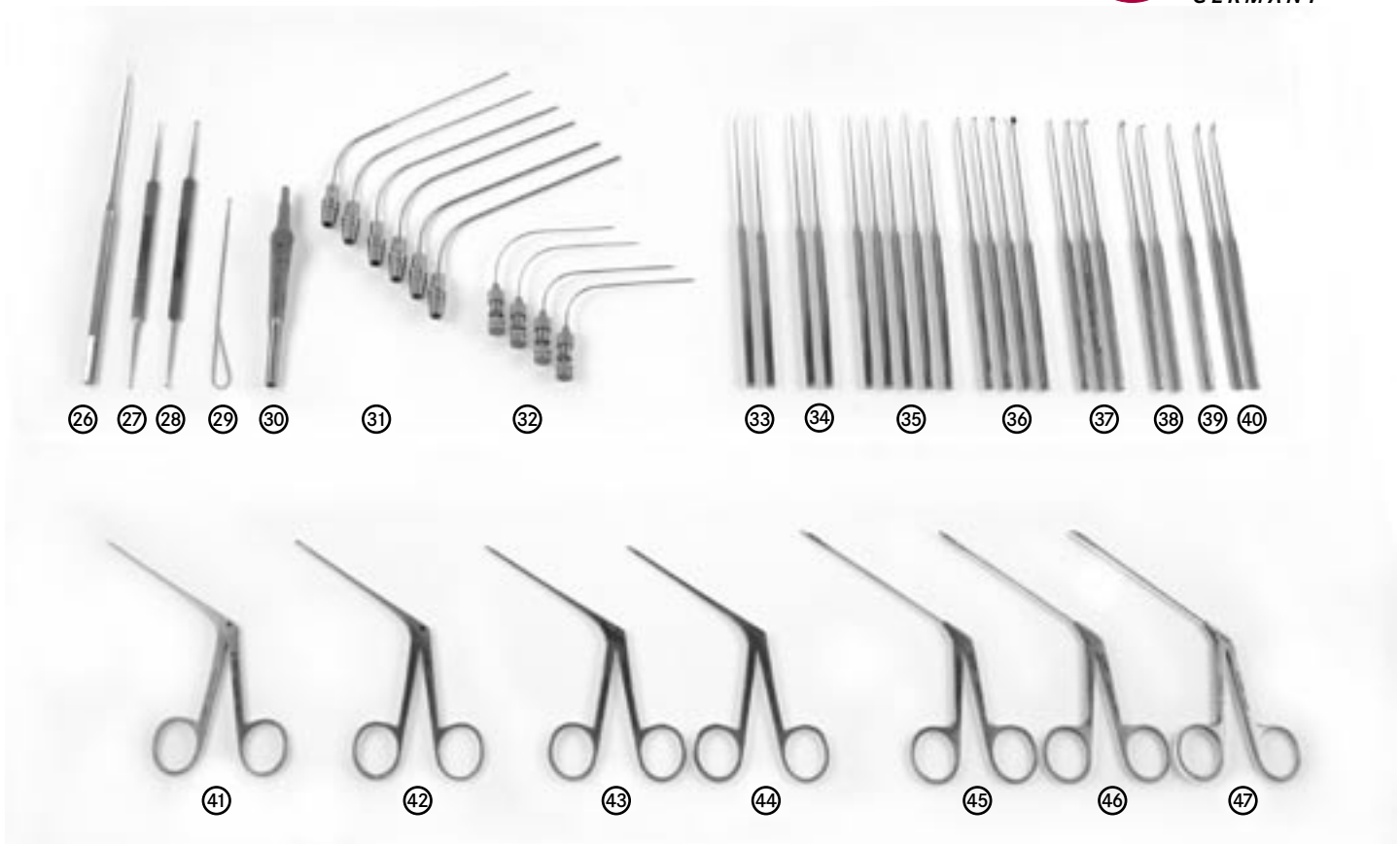
This fine clip is very well suited for holding the ossicle, which is ground to shape as a stapes elevation or columella. The „lock“ helps to prevent the transplant from jumping off.

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The Wigand retractor is well suited for endaural interventions. The sharp part of the retractor is inserted into the incision on the posterior wall of the auditory canal and keeps the soft parts of the pinna as well as the concha away from the view into the auditory canal. The blunt portion displaces the tragus and avoids injury of the latter as is often the case with other, sharp retractors. The intactness of this structure is very advantageous since transplants are often obtained from the tragus area. There is one retractor each for both ear sides. The retractor with the blunt end on the right is used for the left ear, the retractor with the blunt end on the left is used for the right ear.

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The Rosen lancet knife is suited for transecting the anvil stapes connection and the stapedius tendon.

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This forceps has no serration, which could harm the delicate structures of prostheses. It is suited for inserting the prosthesis into the ear.

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The McGee ear forceps is an example of changing standards over the years. With many „McGee forceps“, the front part before the outlet notch for the anvil handle has become too long, with the result that when closing the band or wire ring of the prosthesis, one could abut upon the promontorium. This is a feared manoeuvre in stapes surgery since overly vehement manipulations may lead to hearing loss or breaking in of the footplate. The front part of this forceps has been taken back enabling better working in the tympanum.

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Besides the standard round knife with a 45-degree bend, the bending of the disc knife by 25 degrees has proven very effective in endaural surgery due to the fact of passing through the auditory canal associated with a changed angle of view and direction of work, which is steeper in comparison with the retroauricular approach.

1.0mm / tk 56761-10W
1.5mm / tk 56762-15W
2.5mm / tk 56763-25W
4.0mm / tk 56764-40W

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Tympanoplasty instrument set, as shown on page 4-5 with an additional mini container for the safe storage and sterilization of the micro knives and shambaugh hooks.

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The complete set of all shown instruments and accessories is available under the following Art. code.:

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Stapes instruments and minicontainer of page 8.

tk 56000-05W

STERILISATION - STERILIZATION

Drahtkorb 480x250x70mm

Wire basket 480x250x70mm

tk 87014-03

Sterilisationscontainer
580x280x150mm,
2 Filter, alu, silber

*Sterilization container,
580x280x150mm,
2 filter, aluminium, silver*

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This drilling system consists of a footswitch, which acts as a multifunctional control unit which is able to operate two motors as well as a peristaltic pump. The motor is mainly made of stainless steel. It is working coal- and collector-free. This quality allows the sterilization in an autoclave. This drilling system match the requirements of the MPG.

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©2005 - TEKNO-MEDICAL Optik-Chirurgie GmbH & Co. KG
TEKNO is a registered trademark of Tekno-MEDICAL Optik-Chirurgie GmbH & Co. KG
TEKNO-MEDICAL Optik-Chirurgie GmbH & Co. KG